Tierwaage S230

Beileger Produktinformation

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Warnings

- ▶ In case of malfunction, contact the dealer or the manufacturer. Unauthorised modifications or repairs can damage your scale and void the manufacturer's warranty.
- ▶ Displays must not get wet. Liquids (e.g. water) can damage the display. Use a dry cloth, e.g. a towel, to dry the display.
- ▶ If the scale is not to be used for a long period of time, remove the battery.
- ▶ Avoid stacking materials on the scale or loading the scale with weights permanently. This can cause damage.
- ▶ Place the scale on a firm, stable and level surface to ensure accurate measurement results. The measurement results will change on carpeted floors or if the surface is not level.

Security advice

This scale is designed for professional use.

The user must be familiar with how to operate the scale.

Please read the information provided in the operating instructions carefully before use. It contains important information on installation, proper use and maintenance of the device. The manufacturer is not liable if the following instructions are not observed. The warranty is void if the device is not installed properly. When using electrical components under increased safety requirements, the relevant regulations must be observed.

- ▶ The device is used to determine the weight of animals and objects and may only be used for this purpose. Any other use is prohibited.
- ▶ The pese animaux S230 must not be used in areas where there is a risk of explosion. Areas where there is a risk of explosion can arise through the use of flammable anaesthetics, skin cleansing agents and skin disinfectants.
- ▶ This device is interference-free in accordance with the applicable EU Directive 2014/30/EU. However, the display value may be affected under extreme electrostatic or electromagnetic conditions, e.g. when operating a radio or mobile phone in the immediate vicinity of the device. Once the interference has ceased, the product can be used again as intended; if necessary, it may be necessary to switch it back on. In the event of permanent electrostatic interference, please contact the responsible service partner.
- ▶ The device is a measuring instrument. Drafts, vibrations, rapid temperature changes and sunlight can affect the weighing result.
- ▶ High humidity, vapours, aggressive liquids and heavy soiling should be avoided.

Disposal

Battery and rechargeable battery disposal

Batteries and rechargeable batteries that contain harmful substances are labelled with the symbol of a crossed-out wheelie bin and must not be disposed of in household waste.

As a consumer, you are legally obliged to return used batteries and rechargeable batteries. You can hand in your old batteries and rechargeable batteries as hazardous waste at public collection points in your community or wherever batteries of the type in question are sold. You will find these symbols on batteries that contain harmful substances: Pb = battery contains lead, Cd = battery contains cadmium, Hg = battery contains mercury.

Disposal of the scales

According to current knowledge, the balance does not contain any substances that are particularly hazardous to the environment. The balance should not be treated as normal waste, but must be disposed of at a collection point for the recycling of electrical and electronic equipment.

Further information can be obtained from your local authority or municipal waste disposal company.

Control buttons

ON/OFF/TARE button:

Press the 'ON/OFF/TARE' button to switch on the scale. After 2 seconds, '0.00' appears on the display.

If you place a cloth on the animal scale as a base, press the 'ON/OFF/TARE' button to delete the weight of the cloth. The scale displays '0.00' again and you can now weigh the animal or the object.

Press the 'ON/OFF/TARE' button to switch off the scales.

HOLD button:

You can hold the weight value with the 'HOLD' button.

After the weight value has stabilised, you can record the weight on the display by pressing the 'HOLD' button. If you want to delete the recorded data, press the button again.

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