Babywaage S210

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GB | Baby scale | Instruction manual | page 9

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1. Einführung

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1. Introduction

Thank you for choosing this product from Scaleson UG (limited liability).
It is equipped with all the latest technology and has been optimised for ease of use.

Please read the instructions carefully before use.

If you have any questions or if you encounter problems with your device that are not covered in the instructions, please contact your Scaleson UG (limited liability) service centre or our customer service department:

Telephone: +49 3377 / 9979 395

E-Mail: info@scaleson.net

These instructions describe how to set up and use the Scaleson S210 baby scales.

Scope of delivery:

- ▶ Baby scale
- ▶ Instructions for use

2. Warning notices

- ▶ **Please never leave your baby or toddler unattended on the scale!**
- ▶ In the event of a malfunction, contact the dealer or manufacturer. Unauthorized modifications or repairs may damage the scale and void the manufacturer's warranty.
- ▶ Disconnect the power supply to this scale before any installation, cleaning, or maintenance.
- ▶ If the scale is not going to be used for an extended period of time, please remove the batteries.
- ▶ Avoid stacking materials on the scale or placing excessive weights on it. This can cause damage.
- ▶ Place the scale on a firm, stable, and level surface to ensure accurate measurement results. Measurement results will not be representative if the surface is soft or sloped.
- ▶ Do not connect the scale to unstable power sources.
- ▶ Only use original accessories. Using other brands may cause damage to the terminal.

3. General information

3.1 Technical data

	Baby scale S210
Maximum load:	20 kg
Digit increment:	10 g
Weighing units:	kg/lb/st
Display:	LCD display
Operating temperature:	0° - 40°C
Power supply:	2 x AAA-batteries (not included)
Dimensions of scale:	550 x 320 x 27 mm (L x B x H)

3.2 Intended use

The Scaleson S210 baby scale is designed for weighing babies and small children and is intended for non-calibratable applications.

3. General information

3.3 Safety note

This scale is designed for professional use.

The user must be familiar with the operation of the scale.

Please read the information in the operating instructions carefully before starting operation. It contains important information on the installation, proper use, and maintenance of the device. The manufacturer is not liable if the following instructions are not observed. Improper installation will void the warranty. When using electrical components with increased safety requirements, the relevant regulations must be observed.

- ▶ The device is intended for weighing babies and small children and may only be used for this purpose. Any other use is prohibited.
- ▶ The S210 baby scale must not be used in potentially explosive areas. Potentially explosive areas can arise through the use of flammable anesthetics, skin cleansers, and skin disinfectants.
- ▶ This device is interference-suppressed in accordance with the applicable EU Directive 2014/30/EU. However, extreme electrostatic and electromagnetic influences, e.g., when operating a radio device or mobile phone in the immediate vicinity of the device, can cause interference with the display value. Once the interference has ceased, the product can be used again as intended; it may be necessary to switch it off and on again. In the event of permanent electrostatic interference, please contact the responsible service partner.
- ▶ The device is a measuring instrument. Drafts, vibrations, rapid temperature changes, and sunlight can influence the weighing result.
- ▶ High humidity, vapors, aggressive liquids, and heavy contamination must be avoided.

3.4 Cleaning

A damp cloth and standard cleaning agents are sufficient for cleaning.

Do not use abrasive cleaning agents. Ensure that no liquid penetrates the terminal.

3.5 Maintenance and service

The scale does not require routine maintenance. However, it is advisable to check its accuracy at regular intervals. The frequency depends on the use and condition of the scale. If you notice any deviations, please contact your dealer or the manufacturer.

3. General information

3.6 Warranty / Guarantee / Liability

If there is a defect in the delivered item for which the manufacturer is responsible, the manufacturer is entitled to either remedy the defect or deliver a replacement. Replaced parts become the property of the manufacturer. If the defect cannot be remedied or a replacement cannot be delivered, the statutory provisions apply.

The warranty period is **24 months** and begins on the date of initial purchase of the product.

Please retain the invoice as proof of purchase. In the event of a service call, please contact your dealer or the manufacturer's customer service department.

In particular, no liability is accepted for damage arising from the following reasons: Unsuitable or improper storage or use, faulty assembly or commissioning by the customer or third parties, natural wear and tear, modifications or interventions, faulty or negligent handling, in particular excessive strain, chemical, electrochemical, electrical influences or moisture, unless these are attributable to the manufacturer. If operational, climatic, or other influences lead to a significant change in conditions or the condition of the material, the warranty for the proper overall functioning of the devices shall not apply. The warranty for wear parts (e.g., batteries) is 6 months.

Keep the original packaging for possible return transport!

3.7 Battery and rechargeable battery disposal

Batteries and rechargeable batteries that contain harmful substances are **marked with a crossed-out wheeled bin symbol and must not be disposed of with household waste.**

As a consumer, you are legally obliged to return used batteries and rechargeable batteries. You can dispose of your old batteries and rechargeable batteries as hazardous waste at public collection points in your municipality or anywhere where batteries of the type in question are sold. You will find these symbols on batteries containing harmful substances: Pb = battery contains lead, Cd = battery contains cadmium, Hg = battery contains mercury.

3.8 Disposal of the scale

According to current knowledge, the scale does not contain any substances that are particularly harmful to the environment. The scale should not be treated as normal waste, but must be taken to a collection point for the recycling of electrical and electronic equipment.

Further information is available from your local authority or municipal waste disposal companies.

4. Commissioning

4.1 Installation instructions

Protect the device from shocks, vibrations, extreme heat or cold, drafts, chemicals, and moisture by choosing the right location for installation.

- ▶ Place the scale on a firm, clear, and level surface.
- ▶ Make sure that no cables or other objects are trapped under the scale.

4.2 Power supply

Power is supplied by 2 AAA batteries as standard. Insert these into the battery compartment on the underside of the scale.

5. Operation

5.1 Switch on

ON/OFF/TARE button:

Press the "ON/OFF/TARE" button to turn on the scale. After 2 seconds, "0.00" will appear on the display.

5.2 Switch units

Unit button:

Use the "UNIT" button to switch between the units "kg," "lb," and "st."

You can also use the "Unit" button to hold the weight value.

Once the weight value has stabilized, you can record the weight on the screen by pressing the "UNIT" button. If you want to delete the recorded data, press the button again.

5.3 Taring

If you place a cloth on the baby scale as a base, press the "ON/OFF/TARE" button to delete the weight of the cloth. The scale will display "0.00" again and you can now weigh the baby.

5.4 Weighing

Place the baby in the center of the scale. The weight will appear on the display.

5.6 Switch off / Shut down safely

Press the "ON/OFF/TARE" button to turn off the scale.

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All rights reserved.