# Tierwaage S230

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GB | Baby scale | Instruction manual | page 9

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# 1. Einführung

---

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# 1. Introduction

---

Thank you for choosing this product from Scaleson UG (limited liability).  
It is equipped with all the latest technology and has been optimised for ease of use.

**Please read the instructions carefully before use.**

If you have any questions or if you encounter problems with your device that are not covered in the instructions, please contact your Scaleson UG (limited liability) service centre or our customer service department:

Telephone: +49 3377 / 9979 395

E-Mail: [info@scaleson.net](mailto:info@scaleson.net)

These instructions describe how to set up and use the Scaleson S230 animal scales.

Scope of delivery:

- ▶ Animal scales
- ▶ Rubber mat
- ▶ Insert

# 2. Warning notices

---

- ▶ In the event of a malfunction, contact the retailer or manufacturer. Unauthorised modifications or repairs may damage the scales and invalidate the manufacturer's warranty.
- ▶ Disconnect the power supply to these scales before any installation, cleaning or maintenance.
- ▶ Liquids (e.g. water) may damage the display.
- ▶ Use a dry cloth, e.g. a towel, to dry the display.
- ▶ If the scale is not to be used for a long period of time, please remove the batteries.
- ▶ Avoid stacking materials on the scale or placing excessive weights on it. This can cause damage.
- ▶ Place the scale on a firm, stable and level surface to ensure accurate measurement results. Measurement results will not be representative if the surface is soft or sloped.
- ▶ Only use original accessories. Using other brands may cause damage to the terminal.

### 3. General information

---

#### 3.1 Technical data

	<b>Animal scales S230</b>
Maximum load:	100 kg
Digit increment:	10 g
Weighing units:	kg/lb/lb oz
Display:	LCD display
Power supply:	3 x AAA-batteries (not included)
Dimensions S230.05.100	650 x 450 x 40 mm (L x W x H)
Dimensions S230.15.100	800 x 500 x 40 mm (L x W x H)

#### 3.2 Intended use

The Scaleson S230 animal scale is designed for weighing animals and is intended for non-verifiable applications.

## 3. General information

---

### 3.3 Safety notice

This scale is designed for professional use.  
The user must be familiar with the operation of the scale.

Please read the information in the operating instructions carefully before use. It contains important information on the installation, proper use and maintenance of the device. The manufacturer is not liable if the following instructions are not followed. Improper installation will void the warranty. When using electrical components with increased safety requirements, the relevant regulations must be observed.

- ▶ The device is intended for weighing animals and objects and may only be used for this purpose. Any other use is prohibited.
- ▶ The S230 animal scale must not be used in potentially explosive atmospheres. Potentially explosive atmospheres can arise through the use of flammable anaesthetics, skin cleansers and skin disinfectants.
- ▶ This device is interference-suppressed in accordance with the applicable EU Directive 2014/30/EU. However, extreme electrostatic and electromagnetic influences, e.g. when operating a radio device or mobile phone in the immediate vicinity of the device, can cause the display value to be affected. Once the interference has ceased, the product can be used again as intended; it may be necessary to switch it off and on again. In the event of permanent electrostatic interference, please contact the responsible service partner.
- ▶ The device is a measuring instrument. Draughts, vibrations, rapid temperature changes and sunlight can influence the weighing result.
- ▶ High humidity, vapours, aggressive liquids and heavy soiling should be avoided.

### 3.4 Cleaning

A damp cloth and standard cleaning agents are sufficient for cleaning.  
Do not use abrasive cleaning agents. Ensure that no liquid penetrates the terminal.

### 3.5 Maintenance and service

The scale does not require routine maintenance. However, it is advisable to check its accuracy at regular intervals. The frequency depends on the use and condition of the scale. If you notice any deviations, please contact your dealer or the manufacturer.

## 3. General information

---

### 3.6 Guarantee / Warranty / Liability

If there is a defect in the delivered item for which the manufacturer is responsible, the manufacturer is entitled to either remedy the defect or deliver a replacement. Replaced parts become the property of the manufacturer. If the defect cannot be remedied or a replacement cannot be delivered, the statutory provisions shall apply.

The warranty period is **24 months** and begins on the date of initial purchase of the product.

**Please retain the invoice as proof of purchase.** If servicing is required, please contact your dealer or the manufacturer's customer service department.

**No liability is accepted in particular for damage arising from the following reasons:** Unsuitable or improper storage or use, faulty assembly or commissioning by the customer or third parties, natural wear and tear, modifications or interventions, faulty or negligent handling, in particular excessive strain, chemical, electrochemical or electrical influences or moisture, unless these are attributable to the manufacturer. If operational, climatic or other influences lead to a significant change in conditions or the condition of the material, the warranty for the proper overall functioning of the devices shall lapse. The warranty period for wear parts (e.g. batteries) is 6 months.

**Keep the original packaging for possible return transport!**

### 3.7 Battery and rechargeable battery disposal

Batteries and rechargeable batteries that contain harmful substances are **marked with a crossed-out wheeled bin symbol and must not be disposed of with household waste.**

As a consumer, you are legally obliged to return used batteries and rechargeable batteries. You can dispose of your old batteries and rechargeable batteries as hazardous waste at public collection points in your local authority area or anywhere where batteries of the type in question are sold. You will find these symbols on batteries containing harmful substances: Pb = battery contains lead, Cd = battery contains cadmium, Hg = battery contains mercury.

### 3.8 Disposal of the scales

According to current knowledge, the scales do not contain any substances that are particularly harmful to the environment. The scales should not be treated as normal waste, but must be taken to a collection point for the recycling of electrical and electronic equipment.

Further information is available from your local authority or municipal waste disposal services.

# 4. Commissioning

---

## 4.1 Installation instructions

Protect the device from impact, vibration, extreme heat or cold, draughts, chemicals and moisture by choosing the right location for installation.

- ▶ Place the scales on a firm, clear and level surface.
- ▶ Ensure that no cables or other objects are trapped under the scales.
- ▶ Before use, remove the transport locks on the feet of the scales.

## 4.3 Power supply

Power is supplied by 3 AAA batteries as standard. Insert these into the battery compartment on the underside of the scales.

# 5. Operation

---

## 5.1 Switch on

Press the 'ON/OFF/TARE' button to switch on the scale. After 2 seconds, '0.00' appears on the display.

## 5.2 Switch units

Press the UNIT button to change the unit of measurement to kg, lb or lb oz.

## 5.3 Taring

Place the weight to be tared (e.g. towel or blanket) on the platform and press the ON/OFF/TARE button. The display will return to zero, setting the weight placed on the platform to zero.

Place the animal or object on the scales. Only the weight of the animal or object will be displayed.

To exit tare mode, remove the animal or object and the mat from the scales. A negative weight value will appear on the display. Press the ON/OFF/TARE button to reset the display to zero and delete the tared weight.

# 5. Operation

---

## 5.4 Weighing

Place the animal or object as centrally as possible on the scales. The weight value will appear on the display.

## 5.5 HOLD

You can hold the weight value using the 'HOLD' button.

Once the weight value has stabilised, you can record the weight on the display by pressing the 'HOLD' button. If you want to delete the recorded data, press the button again.

## 5.6 Switch off / Shut down safely

Press and hold the ON/OFF/TARE button to switch off the scales.

To save power, the scales switch off automatically after 2 minutes without any button activity or weight change.

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All rights reserved.