128g
:163×85mm

80g

Shenzhen Jamr Technology Co., Ltd.
A101-301, D101-201, Jamr Science & Technology Park, No. 2
Guiyuan Road, Guixiang Community, Guanlan Street, Longhua
District, 518100 Shenzhen, PEOPLE'S REPUBLIC OF CHINA
Manufacturer item no. / Hersteller-Artikelnr.: BC31LT

Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany

Importer: Scaleson UG (haftungsbeschränkt)
Mellenseestr. 21, 15806 Zossen, Germany.

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

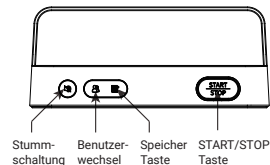
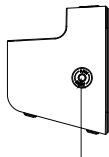
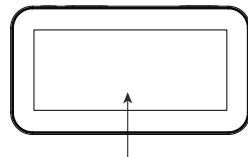
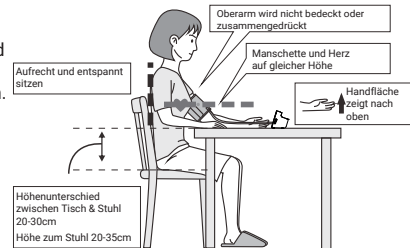
Important Information on Blood Pressure and its Measurement

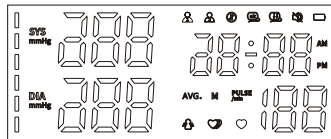The device's blood pressure display consists of a six-part scale. Please refer to the Picture. Different grids represent different interval scales of WHO.


Blood pressure value	WHO grids in device	WHO Classification
DIA<80 & SYS <120	1	Optimal blood pressure
DIA<85 & SYS <130	2	Normal blood pressure
DIA<90 & SYS <140	3	High normal value
DIA<100 & SYS <160	4	Mild hypertension
DIA<110 & SYS <180	5	Moderate hypertension
DIA>=110 or SYS >=180	6	Severe hypertension

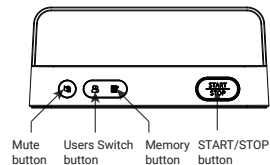
Measuring unit

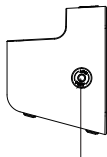
Mute button Users Switch button Memory button START/STOP button

Cuff Connector Port

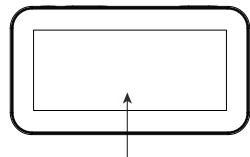LED Display

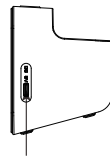AC Adapter Port

Operate on the device

1. Press the START/STOP button to switch on the device; all symbols will appear on the display. The pump will begin to inflate the cuff. The display will continuously show the increasing cuff pressure.

NOTE: If the voice is playing, you can turn off the voice by pressing the () button before the pump begins to inflate the cuff.

2. Once the device has detected your pulse, the heart symbol on the display will start flashing.

3. Once the individual pressure is automatically reached, the pump stops and the pressure drops. The cuff pressure is displayed during the measurement.

4. Once the measurement is complete, the systolic and diastolic blood values and pulse rate are displayed.

5. The measurement results are displayed until you turn the device off by pressing the START/STOP button. If no button is pressed for 60 seconds, the device switches off automatically.

6. Cuff self-checking symbol ()

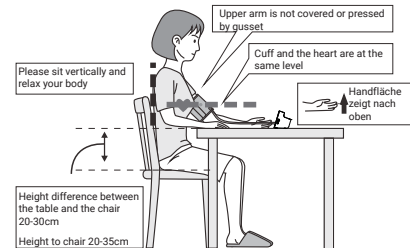
The cuff correct symbol() will be displayed if the cuff position is correct, otherwise the wrong symbol() will be displayed. Please check again the cuff if the wrong symbol() is displayed.

7. Movement error symbol ()

The Movement error symbol () is displayed if you move your body during the measurement. Please remove the cuff, and wait 2-3 minutes. Reapply the cuff and take another measurement.

HINWEIS: Position of the patient:

1. Comfortably seated
2. Legs uncrossed
3. Feet flat on the floor
4. Back and arm supported
5. Middle of the CUFF at the level of the right atrium of the heart

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