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Shenzhen Jamr Technology Co., Ltd.
A101-301, D101-201, Jamr Science & Technology Park, No. 2
Guiyuan Road, Guixiang Community, Guanlan Street, Longhua
District, 518100 Shenzhen, PEOPLE'S REPUBLIC OF CHINA
Manufacturer item no. / Hersteller-Artikelnr.: C02T

Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany

Importer: Scaleson UG (haftungsbeschränkt)
Mellenseestr. 21, 15806 Zossen, Germany

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

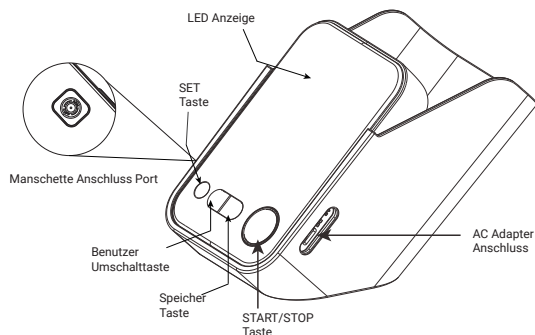
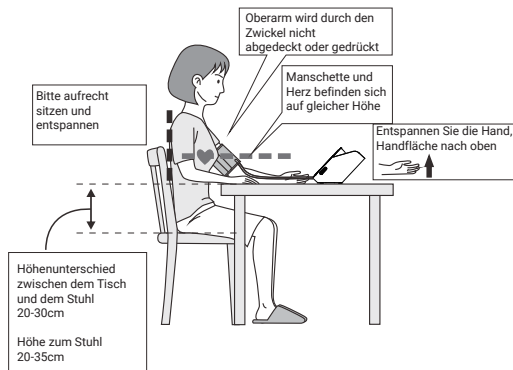
Important Information on Blood Pressure and its Measurement

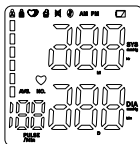The device's blood pressure display consists of a six-part scale. Please refer to the picture. Different grids represent different interval scales of WHO.

Blood pressure value	WHO grids in device	WHO Classification
DIA<80 & SYS <120	1	Optimal blood pressure
DIA<85 & SYS <130	2	Normal blood pressure
DIA<90 & SYS <140	3	High normal value
DIA<100 & SYS <160	4	Mild hypertension
DIA<110 & SYS <180	5	Moderate hypertension
DIA>=110 or SYS >=180	6	Severe hypertension

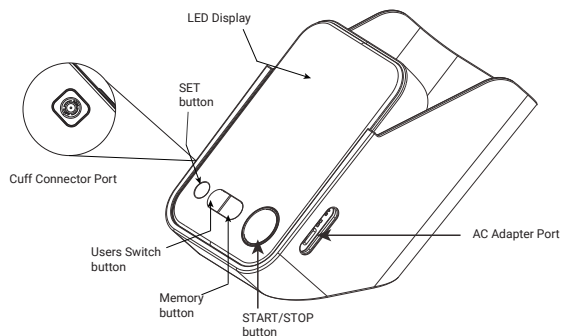Components of your blood pressure monitor


Operate on the device

1. Press the START/STOP button. The pump begins to inflate the cuff. In the display, the increasing cuff pressure is continually displayed.
2. When the device has detected your pulse, the heart symbol in the display begins to blink.
3. When the measurement has been concluded, the measured systolic and diastolic blood pressure values, as well as the pulse will be displayed.
4. The measurement results are displayed until you switch the device off. If no button is pressed for 60 seconds, the device switches off automatically.

NOTE:

Patient Position:

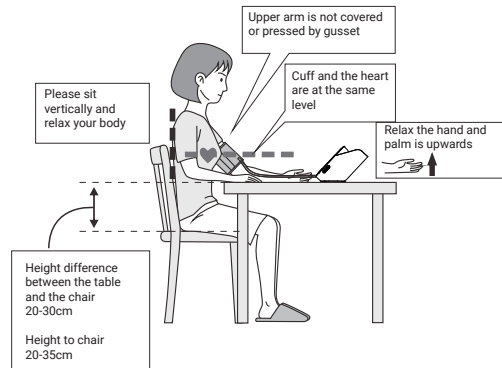1. Comfortably seated
2. Legs uncrossed
3. Feet flat on the floor
4. Back and arm supported
5. Middle of the CUFF at the level of the right atrium of the heart

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